


## INSTRUCTIONS FOR USE

### DISPOSABLE BIPOLAR GRASPING FORCEPS

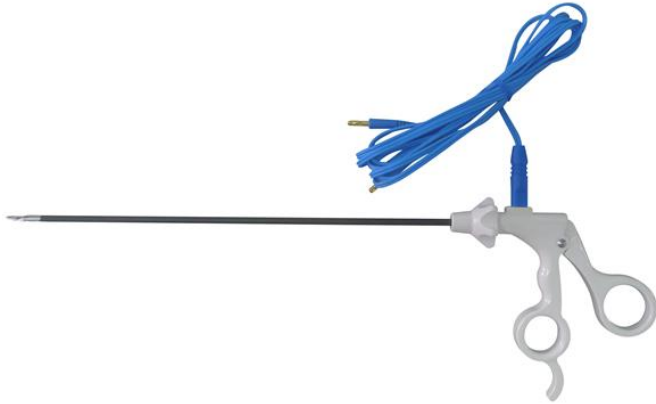

**IMPORTANT** Please read instruction carefully before use.

These disposable forceps should only be used by trained surgical staff. These instructions are for general use, and do not represent instructions for any procedures during which the products may be used.

#### PRODUCT DESCRIPTION

Disposable Bipolar Grasping Forceps presented in sterile medical-grade packaging. The product is designed for laparoscopic surgery and has a 5 x 330mm insulated cannula (*black*), with stainless steel grasping tips and a white polypropylene handle with connector. Product is supplied with a 3m single-use bipolar cable with ValleyLab 2-pin plug and European block connector.

Product is supplied 1pcs/blister, 5pcs/box and Sterile.

#### INDICATION

Any laparoscopic procedure where electrosurgical grasping forceps are required. Product should be used with associated electrosurgical devices to ensure product/patient safety (*e.g. approved high-frequency generators, grounding pads etc.*).

#### PRODUCT INTEGRITY

Product should be presented with smooth, black insulated cannula, clean stainless steel tips and intact white handle. If any of these characteristics are not as described (*or if obvious aesthetic changes are evident*), product should not be used and supplier advised.

Electrode connections should be secure. Loose fitting of cables may cause product to malfunction.

#### PRECAUTIONS

Power supply must be  $\leq 50W$ . Excess of this may cause product damage and malfunction. For use with approved high-frequency generators only.

Max. repeat-peak voltage 400v

Product gets hot and should not be placed on flammable surfaces where risk of burns or fire exist.

Avoid electrode contact with other metal surfaces during use. Power supply should be switched off before electrode is placed down.

When disconnecting cable from high frequency generator, grasp plug rather than cable.

## BEFORE USE












Ensure packaging integrity. Ensure product is free from defects such as uneven insulation, rough edges, loose parts etc.

## CONTRAINDICATIONS

There are no contraindications for this product, however, the device should be used with appropriate power suppliers only to ensure proper function.

## PACKAGING & HANDLING

Symbols used in packaging

 Avoid direct sunlight	 Store below 30°C	 Store in cool, dry place
		 Single Use Only
 Expiry Date (2yrs)	 Do not use if packaging damaged	
 Manufacturer	 Stapleline Medizintechnik GmbH Universitätsstr. 90, Bochum44789, Germany	

Product should be stored out of direct sunlight and below 30 degrees celsius.